



NeoMed® Oral Dispenser in Pharmacy Pack
For Oral Use Only
Instructions for Use

Indications for Use:

The device is indicated for use as a dispenser, a measuring device and an oral fluid transfer device.

Contraindications:

The NeoMed Oral Dispenser is intended for one-time use. Each dispenser should be used only for one dispensing and one patient.

WARNING:

Rapid dispensing of liquid orally may cause choking. Dispense slowly, a small amount at a time, to allow time for swallowing. Tip caps are a choking hazard. Keep out of reach of children.

Directions:

1. Follow physician's instructions or hospital protocol in opening package and use of contents. This device should be opened in an aseptic environment.
2. Fully depress the plunger before filling dispenser. Place the tip into the liquid and draw liquid into dispenser slowly. For accurate measurement, use the black leading edge of the gasket toward the plunger tip.
3. To expel excess air bubbles, hold dispenser with tip pointing up. Tap side of dispenser to cause bubbles to rise toward tip. Draw back slightly on plunger, then push plunger upward to eject air in barrel and tip. Repeat until bubbles are eliminated.
4. Confirm dose accuracy. Place the orange cap on the dispenser and label until ready to use (label not included).
5. Liquid should be administered via enteral-only connectors or directly into mouth (large pediatric or older patients). This dispenser is not designed for use with luer or other injection connections. If dispensing directly into mouth, ensure patient is upright before giving liquid. Direct dispenser tip towards inside of cheek.
6. Flush the dispenser with water and dispose in closed plastics or dispose as required by hospital protocols and local or national regulations. Do not autoclave or re-use dispenser as this may increase risk of leakage or biological infection.

How Supplied:

NON-STERILE					STERILE		
Size	Clear with Orange Gradients	Clear with Blue Gradients	Amber with White Gradients	Quantity / Case	Size	Amber with White Gradients	Quantity / Case
0.5 mL	BC-S05EO	BB-S05EO	BA-S05EO	500	1 mL	BA-S1EO-S	200
1 mL	BC-S1EO	BB-S1EO	BA-S1EO	500	3 mL	BA-S3EO-S	200
3 mL	BC-S3EO	BB-S3EO	BA-S3EO	500	6 mL	BA-S6EO-S	200
6 mL	BC-S6EO	BB-S6EO	BA-S6EO	500	12 mL	BA-12EO-S	200
12 mL	BC-S12EO	BB-S12EO	BA-S12EO	500			
20 mL	BC-S20EO	BB-S20EO	BA-S20EO	200			
35 mL	BC-S35EO	BB-S35EO	BA-S35EO	200			
60 mL	BC-S60EO	BB-S60EO	BA-S60EO	200			

Symbols used on Labels:

 Manufacturer Fabricant Fabricante	 Consult instructions for use Consulter mode d'emploi Consultar instrucciones de uso Consulte instruções de utilização	 Use By (Date) A utiliser avant (Date) Usar antes de (Fecha) Utilizar em (Data)
 Non-Sterile Non-stérile No Estéril Não-estéril	 Catalog Number Numéro de catalogue Número de catálogo Número de catálogo	 Sterilized Using Ethylene Oxide Méthode de stérilisation : oxyde d'éthylène Método de esterilización: óxido de etileno Método de esterilização: óxido de etileno
 Lot Number Numéro de lot Número de lote Número de lote	 Do Not Use if Package is Damaged Ne pas utiliser si l'emballage est endommagé Não usar se o embalagem estiver danificada No usar si el paquete está dañado	
 Do Not Re-use Ne Réutiliser No Reutilice Não Reutilize	 Authorized Representative in the European Community Représentant agréé pour la C.E.E. Representante autorizado en la Comunidad Europea Representante autorizado na União Europeia	

This device is not made with natural rubber latex, DEHP or BPA.

 Manufactured by NeoMed Incorporated,
100 Londonderry Ct, Suite 112, Woodstock, GA 30188 USA
(888) 876-2225; www.neomedinc.com
Emergo Europe, Molenstraat 15, 2513 BH,
The Hague, The Netherlands



Australian Sponsor: Emergo Australia
Level 20, Tower II Darling Park
201 Sussex Street, Sydney, NSW 2000
Australia

NeoMed and the NeoMed logo are registered trademarks of NeoMed, Inc. © 2015 NeoMed, Inc.

LBL-OD-009 Rev 8 Effective: 05-2015